

# Respireo **SOFT** Nasal Silent

A peaceful sleep  
for both of you



Equipped with the new **PEACE SYSTEM**.  
An innovative exhalation port allowing for a quieter and more diffuse air flow.

## *Peaceful nights*

- 26,3% quieter than Respireo SOFT Nasal

\*acoustic pressure measured in dB(A) in the anechoic chamber of the Materiacustica Lab - University of Ferrara.

## *Excellent diffusion*

- your partner will have a peaceful sleep, thanks to the optimal diffusion of the exhalation flow



**The Univeo headgear**  
is designed to self-locate high on the head in order to distribute the support and offer a greater stability

**Light and durable frame**  
The new generation frame is improved for an excellent durability, lightness and unobstructed vision

**Respireo SOFT cushion**  
The new improved version of the cushion allows for an even better fit at the nasal bridge, and a delicate effective seal.

**Diffused and quiet venting with Peace System**  
The new vent port ensures a quieter flow and a more efficient diffusion






**Ergonomic hooks**  
Making easy for patients to remove the mask and lock it in the same fit

**360° rotating elbow**  
With a quick release swivel for freedom of movements

**Flexible lateral arms**  
Auto-adjust to facial width for a perfect stability without requiring any tension of the headgear



### Fitting guide

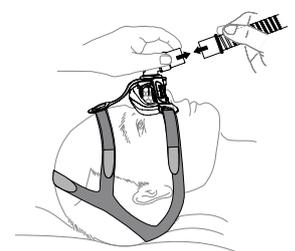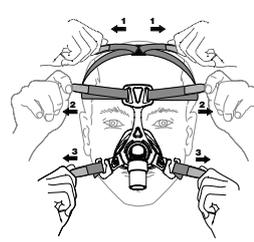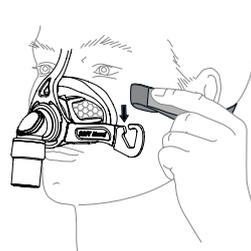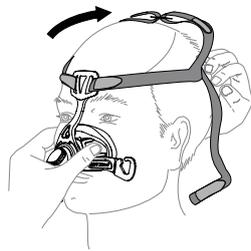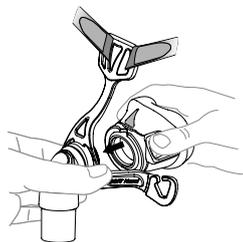

**1** Align the two triangles and clipse the cushion on the frame.  
  
Tips: The two lines under the cushion and frame should also be lined up.

**2** Position the mask on the face (upper part of the cushion at 2/3 of the nose) while pulling the headgear over the head.

**3** Loop the lower straps of the headgear into the hooks.

**4** Adjust the settings of the headgear from top to down (1, 2 then 3).  
  
Tips: Ensure to not tight too much the headgear.

**5** Make final adjustment while lying down. Softly rise the cushion until it naturally sits on the bridge of the nose. If needed, gently release/tighten the headgear settings.

Note: For complete user instructions refer to the User Manual of the device. Find also the fitting guide directly on YouTube to know how to install, adjust and clean your Respireo SOFT Nasal Silent.



Codes	Designation	Size
KM229106	Respireo SOFT Nasal Silent (manufacturer ref. 472365*)	M
KM229107	Respireo SOFT Nasal Silent (manufacturer ref. 472366*)	L
KM227700	Univeo headgear (manufacturer reference 472471**)	
KM227800	Respireo SOFT Nasal Cushion (manufacturer reference 472470)	M
KM228700	Respireo SOFT Nasal Cushion (manufacturer reference 472473)	L