



## A single step for simplicity and safety

Respireo™ Hospital F2 is a range of single-use\* masks designed to provide respiratory support for hospital patients.

### 4 adjustment points headgear

Designed to be easy to set-up and to fit all head morphologies

### Silicone double-wall cushion

Provides a comfortable and optimal seal by adapting to the patient's face

### Quick-release clips

Designed for easy headgear attachment and quick release in case of emergency



### Silicone forehead pad

Designed to ensure mask stability and patient comfort

### Transparent frame

Designed to provide an optimal line of sight for the clinicians and patients

### 360° rotating elbow and interchangeable

Facilitates the movement and the position of the tube

## Modularity and simplicity

- 3 versions available (non vented, vented and non vented with anti-asphyxia valve)
- Cushion available in 3 sizes (S, M, L)
- **3 types of interchangeable elbows** to facilitate patient management when changing therapy and equipment



Non vented



Vented



Non vented with anti-asphyxia valve

- Easy to set-up

\* For up to 7 days.

### Masks references

Product reference	Description	Size	Distributor code
472570	Respireo™ Hospital F2 Vented Disposable - A	S	KM229194
472571	Respireo™ Hospital F2 Vented Disposable - A	M	KM229195
472572	Respireo™ Hospital F2 Vented Disposable - A	L	KM229196
472641	Respireo™ Hospital F2 Vented Disposable - B	S	KM229209
472642	Respireo™ Hospital F2 Vented Disposable - B	M	KM229210
472643	Respireo™ Hospital F2 Vented Disposable - B	L	KM229211
472573	Respireo™ Hospital F2 Non Vented Disposable - A	S	KM229200
472574	Respireo™ Hospital F2 Non Vented Disposable - A	M	KM229201
472575	Respireo™ Hospital F2 Non Vented Disposable - A	L	KM229202
472644	Respireo™ Hospital F2 Non Vented Disposable - B	S	KM229206
472645	Respireo™ Hospital F2 Non Vented Disposable - B	M	KM229207
472646	Respireo™ Hospital F2 Non Vented Disposable - B	L	KM229208
472576	Respireo™ Hospital F2 Non Vented Disposable with anti-asphyxia valve - A	S	KM229197
472577	Respireo™ Hospital F2 Non Vented Disposable with anti-asphyxia valve - A	M	KM229198
472578	Respireo™ Hospital F2 Non Vented Disposable with anti-asphyxia valve - A	L	KM229199
472647	Respireo™ Hospital F2 Non Vented Disposable with anti-asphyxia valve - B	S	KM229203
472648	Respireo™ Hospital F2 Non Vented Disposable with anti-asphyxia valve - B	M	KM229204
472649	Respireo™ Hospital F2 Non Vented Disposable with anti-asphyxia valve - B	L	KM229205

To find out the right reference (A or B) for your country, please contact our sales teams.

### Interchangeable elbows references

Product reference	Description	Distributor code
472412	Respireo™ Blue Non Vented elbow	KM229109
472627	Respireo™ Hospital Vented elbow	KM229212
472628	Respireo™ Hospital Non Vented with anti-asphyxia valve elbow	KM229213

The above listed elbows are CE marked as medical devices, and they are used to change the intended use of the masks on which they will be mounted on. Please read the respective instructions for use.

#### Manufactured by

AIR LIQUIDE MEDICAL SYSTEMS S.r.l  
Via Bisceglie, 66 - 20152 Milano (MI) - ITALY  
PLANT: Via dei Prati, 62 - 25073 Bovezzo (BS) - ITALY  
Tel. +39 030.201.59.11  
Fax +39 030.209.83.29

#### Distributor Contact

AIR LIQUIDE MEDICAL SYSTEMS S.A.  
Parc de Haute Technologie  
6, rue Georges Besse  
92182 Antony Cedex  
France

medicaldevice.airliquide.com

 **Hotline ALMS: +33 179 51 70 01**



Respireo™ Hospital F2 Non Vented Disposable is an oronasal mask without calibrated exhalation orifices, intended to be used for short time periods (less than 7 days) in a hospital setting by a single adult patient (weighing over 30 kg) in controlled non-invasive positive pressure ventilation (NIPPV).

Respireo™ Hospital F2 Vented Disposable is an oronasal mask with calibrated exhalation orifices, intended to be used for short time periods (less than 7 days) in a hospital setting by a single adult patient (weighing over 30 kg) who have been prescribed for a non-invasive positive pressure ventilation therapy (NIPPV), e.g. continuous positive airway pressure therapy (CPAP) or Bilevel positive airway pressure therapy (BiPAP).

Respireo™ Hospital F2 Non Vented Disposable with anti-asphyxia valve is an oronasal mask without calibrated exhalation orifices and with an anti-suffocation valve, intended to be used for short time periods (less than 7 days) in a hospital setting by a single adult patient (weighing over 30 kg) in controlled non-invasive positive pressure ventilation (NIPPV).

Medical device class IIa CE0051. Please read the instructions for use.