FRANÇAIS

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Air Liquide Medical Systems S.r.l.
Département technique

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ENGLISH

Decontamination, disinfection and sterilization guide for masks.

This guide defines the procedures validated by Air Liquide Medical Systems S.r.l. to be performed only in hospital or clinical structures for the regeneration of the masks in order to make them suitable to be reused in compliance with UNI EN ISO 17664:2021. It is up to the person in charge of the regeneration processes to ensure that the operations are performed using suitable materials, devices and personnel in order to achieve the desired result. This generally requires validation and systematic monitoring of the entire process. The manufacturer does not respond in case the process does not comply with the instructions provided, both for its actual effectiveness or any possible undesirable consequences.

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* For Respireo Hospital F Non Vented Reusable, in order to perform a complete cleaning of the regulation screw is necessary to completely rotate counterclockwise the knob.

** For Respireo SOFT Baby, Ineva SOFT Baby, Alnest SOFT Baby and Respireo SOFT Child, Ineva SOFT Child, Alnest SOFT Child, in both Vented and Non Vented versions, the tube assembly cannot be disinfected nor sterilized.

PHASE 1	PREPARATION	Disassemble entirely the mask according to the instructions for use of the specific device. The headgear is not suitable for regeneration, so it shall be replaced.
PHASE 2	DECONTAMINATION	<p>Perform DECONTAMINATION always and with care before choosing between THERMAL DISINFECTION or CHEMICAL DISINFECTION.</p> <ul style="list-style-type: none"> Soak the components for at least 15 minutes in water containing a neutral pH detergent (i.e. gigazyme[®] with 1% dilution) then clean each of them for one minute with a soft-bristle brush. Rinse all the components thoroughly in running tap water. Air-dry the components away from direct sunlight.
PHASE 3A	HIGH LEVEL THERMAL DISINFECTION	<ul style="list-style-type: none"> Using a certified automatic hot water disinfection system, soak the components in a solution of a neutral pH detergent, in combination with a neutral pH rinsing agent if deemed appropriate. In compliance with UNI EN ISO 15883:2014, the cleaning cycle must comply with one of the following temperature-time combinations: 70°C - 100 minutes 80°C - 10 minutes 90°C - 1 minute Air-dry the components away from direct sunlight.
PHASE 3B	HIGH LEVEL CHEMICAL DISINFECTION	<ul style="list-style-type: none"> Soak the components in one of the following high-level disinfectant solutions, following the instructions provided by the solution producer: - CIDEX[®] OPA Rinse the components thoroughly in tap water for at least one minute. Perform this operation 3 times, using clean water each time. Air-dry the components away from direct sunlight.
PHASE 4	AUTOCLAVE STERILIZATION	<ul style="list-style-type: none"> Sterilise the components in an autoclave at 121°C (250°F) for no longer than 30 minutes. Drying takes place during the sterilization process.
PHASE 5	INSPECTION	<p>Visually inspect each part of the mask.</p> <p>Note: If any component shows signs of deterioration (e.g. cracks, scratches, fractures or tears), it should be discarded and replaced. Slight discolouration of the silicone components is quite normal.</p>
PHASE 6	REASSEMBLING	Reassemble the mask according to the instructions for use of the specific device.
PHASE 7	CONSERVATION	Store in a dry and dust free environment at temperature between -25°C and +50°C, away from direct sunlight.

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Air Liquide Medical Systems S.r.l.
Technical department

MASKE	DEKONTAMINERING	TERMISK DESINFEKTION	KEMISK DESINFEKTION	STERILISERING	HOMOLOGEREDE CYKLUSSE
Respireo N	✓	I.R.	I.R.	I.R.	I.R.
Respireo Primo N	✓	I.R.	I.R.	I.R.	I.R.
Respireo Primo N Non Vented	✓	✓	✓	✓	20
Respireo Primo F	✓	I.R.	I.R.	I.R.	I.R.
Respireo Primo F Non Vented	✓	I.R.	I.R.	I.R.	I.R.
Respireo Hospital F Vented Disposable	I.R.	I.R.	I.R.	I.R.	I.R.
Respireo Hospital F Non Vented Disposable	I.R.	I.R.	I.R.	I.R.	I.R.
Respireo Hospital F Non Vented Disposable with anti-asphyxia valve	I.R.	I.R.	I.R.	I.R.	I.R.
Respireo Hospital F Non Vented Reusable *	✓	✓	✓	✓	20
Respireo Blue Non Vented elbow	✓	✓	✓	✓	20
Respireo Primo P	I.R.	I.R.	I.R.	I.R.	I.R.
Respireo Primo P Non Vented	I.R.	I.R.	I.R.	I.R.	I.R.
Respireo SOFT Baby **	✓	✓	✓	✓	12
Ineva SOFT Baby **	✓	✓	✓	✓	12
Respireo SOFT Baby Non Vented **	✓	✓	✓	✓	12
Ineva SOFT Baby Non Vented **	✓	✓	✓	✓	12
Respireo SOFT Child **	✓	✓	✓	✓	12
Ineva SOFT Child **	✓	✓	✓	✓	12
Respireo SOFT Child Non Vented **	✓	✓	✓	✓	12
Ineva SOFT Child Non Vented **	✓	✓	✓	✓	12
Respireo SOFT Nasal	✓	✓	✓	I.R.	20

Respireo SOFT Nasal Silent	✓	✓	✓	I.R.	20
Nasal Non Vented elbow	✓	✓	✓	✓	20
Alnest N1 Silent	✓	✓	✓	I.R.	20
Ineva N1 Silent	✓	✓	✓	I.R.	20
Alnest N1	✓	✓	✓	I.R.	20
Ineva N1	✓	✓	✓	I.R.	20
Respireo SOFT Facial	✓	✓	✓	I.R.	20
Respireo SOFT Facial Non Vented	✓	✓	✓	✓	20
Alnest MC1	I.R.	I.R.	I.R.	I.R.	I.R.
Ineva MC1	I.R.	I.R.	I.R.	I.R.	I.R.
Respireo Hospital F2 Vented Disposable	I.R.	I.R.	I.R.	I.R.	I.R.
Respireo Hospital F2 Non Vented Disposable	I.R.	I.R.	I.R.	I.R.	I.R.
Respireo Hospital F2 Non Vented Disposable with anti-asphyxia valve	I.R.	I.R.	I.R.	I.R.	I.R.
Respireo Hospital Vented elbow	I.R.	I.R.	I.R.	I.R.	I.R.
Respireo Hospital Non Vented with anti-asphyxia valve elbow	I.R.	I.R.	I.R.	I.R.	I.R.
Alnest SOFT Baby **	✓	✓	✓	✓	12
Alnest SOFT Baby Non Vented **	✓	✓	✓	✓	12
Alnest SOFT Child **	✓	✓	✓	✓	12
Alnest SOFT Child Non Vented **	✓	✓	✓	✓	12

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Air Liquide Medical Systems S.r.l.
Teknisk avdeling

MASKEN	DEKONTAMINERING	TERMISK DESINFEKSJON	KJEMISK DESINFEKSJON	STERILISERING	GODKJENTE SYKLUSER
Respireo N	✓	I.R.	I.R.	I.R.	I.R.
Respireo Primo N	✓	I.R.	I.R.	I.R.	I.R.
Respireo Primo N Non Vented	✓	✓	✓	✓	20
Respireo Primo F	✓	I.R.	I.R.	I.R.	I.R.
Respireo Primo F Non Vented	✓	I.R.	I.R.	I.R.	I.R.
Respireo Hospital F Vented Disposable	I.R.	I.R.	I.R.	I.R.	I.R.
Respireo Hospital F Non Vented Disposable	I.R.	I.R.	I.R.	I.R.	I.R.
Respireo Hospital F Non Vented Disposable with anti-asphyxia valve	I.R.	I.R.	I.R.	I.R.	I.R.
Respireo Hospital F Non Vented Reusable *	✓	✓	✓	✓	20
Respireo Blue Non Vented elbow	✓	✓	✓	✓	20
Respireo Primo P	I.R.	I.R.	I.R.	I.R.	I.R.
Respireo Primo P Non Vented	I.R.	I.R.	I.R.	I.R.	I.R.
Respireo SOFT Baby **	✓	✓	✓	✓	12
Ineva SOFT Baby **	✓	✓	✓	✓	12
Respireo SOFT Baby Non Vented **	✓	✓	✓	✓	12
Ineva SOFT Baby Non Vented **	✓	✓	✓	✓	12
Respireo SOFT Child **	✓	✓	✓	✓	12
Ineva SOFT Child **	✓	✓	✓	✓	12
Respireo SOFT Child Non Vented **	✓	✓	✓	✓	12
Ineva SOFT Child Non Vented **	✓	✓	✓	✓	12
Respireo SOFT Nasal	✓	✓	✓	I.R.	20

Respireo SOFT Nasal Silent	✓	✓	✓	I.R.	20
Nasal Non Vented elbow	✓	✓	✓	✓	20
Alnest N1 Silent	✓	✓	✓	I.R.	20
Ineva N1 Silent	✓	✓	✓	I.R.	20
Alnest N1	✓	✓	✓	I.R.	20
Ineva N1	✓	✓	✓	I.R.	20
Respireo SOFT Facial	✓	✓	✓	I.R.	20
Respireo SOFT Facial Non Vented	✓	✓	✓	✓	20
Alnest MC1	I.R.	I.R.	I.R.	I.R.	I.R.
Ineva MC1	I.R.	I.R.	I.R.	I.R.	I.R.
Respireo Hospital F2 Vented Disposable	I.R.	I.R.	I.R.	I.R.	I.R.
Respireo Hospital F2 Non Vented Disposable	I.R.	I.R.	I.R.	I.R.	I.R.
Respireo Hospital F2 Non Vented Disposable with anti-asphyxia valve	I.R.	I.R.	I.R.	I.R.	I.R.
Respireo Hospital Vented elbow	I.R.	I.R.	I.R.	I.R.	I.R.
Respireo Hospital Non Vented with anti-asphyxia valve elbow	I.R.	I.R.	I.R.	I.R.	I.R.
Alnest SOFT Baby **	✓	✓	✓	✓	12
Alnest SOFT Baby Non Vented **	✓	✓	✓	✓	12
Alnest SOFT Child **	✓	✓	✓	✓	12
Alnest SOFT Child Non Vented **	✓	✓	✓	✓	12

* For Respireo Hospital F Non Vented Reusable, er det nødvendig at knotten dreies helt mot klokken for at justeringskruen skal bli helt ren.

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Air Liquide Medical Systems S.r.l.
Technical department

MASK	SANERING	VÄRMEDESINFEKTION	KEMISK DESINFEKTION	STERILISERING	TYPGODKÄNDA CYKLER
Respireo N	✓	N.A.	N.A.	N.A.	N.A.
Respireo Primo N	✓	N.A.	N.A.	N.A.	N.A.
Respireo Primo N Non Vented	✓	✓	✓	✓	20
Respireo Primo F	✓	N.A.	N.A.	N.A.	N.A.
Respireo Primo F Non Vented	✓	N.A.	N.A.	N.A.	N.A.
Respireo Hospital F Vented Disposable	N.A.	N.A.	N.A.	N.A.	N.A.
Respireo Hospital F Non Vented Disposable	N.A.	N.A.	N.A.	N.A.	N.A.
Respireo Hospital F Non Vented Disposable with anti-asphyxia valve	N.A.	N.A.	N.A.	N.A.	N.A.
Respireo Hospital F Non Vented Reusable *	✓	✓	✓	✓	20
Respireo Blue Non Vented elbow	✓	✓	✓	✓	20
Respireo Primo P	N.A.	N.A.	N.A.	N.A.	N.A.
Respireo Primo P Non Vented	N.A.	N.A.	N.A.	N.A.	N.A.
Respireo SOFT Baby **	✓	✓	✓	✓	12
Ineva SOFT Baby **	✓	✓	✓	✓	12
Respireo SOFT Baby Non Vented **	✓	✓	✓	✓	12
Ineva SOFT Baby Non Vented **	✓	✓	✓	✓	12
Respireo SOFT Child **	✓	✓	✓	✓	12
Ineva SOFT Child **	✓	✓	✓	✓	12
Respireo SOFT Child Non Vented **	✓	✓	✓	✓	12
Ineva SOFT Child Non Vented **	✓	✓	✓	✓	12
Respireo SOFT Nasal	✓	✓	✓	N.A.	20

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MASKA	ODKAŻANIE	DEZYNFEKCJA TERMICZNA	DEZYNFEKCJA CHEMICZNA	STERYLIZACJA	CYKLE HOMOLOGOWANE
Respireo N	✓	N.D.	N.D.	N.D.	N.D.
Respireo Primo N	✓	N.D.	N.D.	N.D.	N.D.
Respireo Primo N Non Vented	✓	✓	✓	✓	20
Respireo Primo F	✓	N.D.	N.D.	N.D.	N.D.
Respireo Primo F Non Vented	✓	N.D.	N.D.	N.D.	N.D.
Respireo Hospital F Vented Disposable	N.D.	N.D.	N.D.	N.D.	N.D.
Respireo Hospital F Non Vented Disposable	N.D.	N.D.	N.D.	N.D.	N.D.
Respireo Hospital F Non Vented Disposable with anti-asphyxia valve	N.D.	N.D.	N.D.	N.D.	N.D.
Respireo Hospital F Non Vented Reusable *	✓	✓	✓	✓	20
Respireo Blue Non Vented elbow	✓	✓	✓	✓	20
Respireo Primo P	N.D.	N.D.	N.D.	N.D.	N.D.
Respireo Primo P Non Vented	N.D.	N.D.	N.D.	N.D.	N.D.
Respireo SOFT Baby **	✓	✓	✓	✓	12
Ineva SOFT Baby **	✓	✓	✓	✓	12
Respireo SOFT Baby Non Vented **	✓	✓	✓	✓	12
Ineva SOFT Baby Non Vented **	✓	✓	✓	✓	12
Respireo SOFT Child **	✓	✓	✓	✓	12
Ineva SOFT Child **	✓	✓	✓	✓	12
Respireo SOFT Child Non Vented **	✓	✓	✓	✓	12
Ineva SOFT Child Non Vented **	✓	✓	✓	✓	12
Respireo SOFT Nasal	✓	✓	✓	N.D.	20

Respireo SOFT Nasal Silent	✓	✓	✓	N.D.	20
Nasal Non Vented elbow	✓	✓	✓	✓	20
Alnest N1 Silent	✓	✓	✓	N.D.	20
Ineva N1 Silent	✓	✓	✓	N.D.	20
Alnest N1	✓	✓	✓	N.D.	20
Ineva N1	✓	✓	✓	N.D.	20
Respireo SOFT Facial	✓	✓	✓	N.D.	20
Respireo SOFT Facial Non Vented	✓	✓	✓	✓	20
Alnest MC1	N.D.	N.D.	N.D.	N.D.	N.D.
Ineva MC1	N.D.	N.D.	N.D.	N.D.	N.D.
Respireo Hospital F2 Vented Disposable	N.D.	N.D.	N.D.	N.D.	N.D.
Respireo Hospital F2 Non Vented Disposable	N.D.	N.D.	N.D.	N.D.	N.D.
Respireo Hospital F2 Non Vented Disposable with anti-asphyxia valve	N.D.	N.D.	N.D.	N.D.	N.D.
Respireo Hospital Vented elbow	N.D.	N.D.	N.D.	N.D.	N.D.
Respireo Hospital Non Vented with anti-asphyxia valve elbow	N.D.	N.D.	N.D.	N.D.	N.D.
Alnest SOFT Baby **	✓	✓	✓	✓	12
Alnest SOFT Baby Non Vented **	✓	✓	✓	✓	12
Alnest SOFT Child **	✓	✓	✓	✓	12
Alnest SOFT Child Non Vented **	✓	✓	✓	✓	12

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MASZK	TISZTÍTÁS	HŐFERTŐTLENÍTÉS	VEGYSZERES FERTŐTLENÍTÉS	STERILIZÁLÁS	JÓVÁHAGYOTT CIKLUSOK
Respireo N	✓	N.A.	N.A.	N.A.	N.A.
Respireo Primo N	✓	N.A.	N.A.	N.A.	N.A.
Respireo Primo N Non Vented	✓	✓	✓	✓	20
Respireo Primo F	✓	N.A.	N.A.	N.A.	N.A.
Respireo Primo F Non Vented	✓	N.A.	N.A.	N.A.	N.A.
Respireo Hospital F Vented Disposable	N.A.	N.A.	N.A.	N.A.	N.A.
Respireo Hospital F Non Vented Disposable	N.A.	N.A.	N.A.	N.A.	N.A.
Respireo Hospital F Non Vented Disposable with anti-asphyxia valve	N.A.	N.A.	N.A.	N.A.	N.A.
Respireo Hospital F Non Vented Reusable *	✓	✓	✓	✓	20
Respireo Blue Non Vented elbow	✓	✓	✓	✓	20
Respireo Primo P	N.A.	N.A.	N.A.	N.A.	N.A.
Respireo Primo P Non Vented	N.A.	N.A.	N.A.	N.A.	N.A.
Respireo SOFT Baby **	✓	✓	✓	✓	12
Ineva SOFT Baby **	✓	✓	✓	✓	12
Respireo SOFT Baby Non Vented **	✓	✓	✓	✓	12
Ineva SOFT Baby Non Vented **	✓	✓	✓	✓	12
Respireo SOFT Child **	✓	✓	✓	✓	12
Ineva SOFT Child **	✓	✓	✓	✓	12
Respireo SOFT Child Non Vented **	✓	✓	✓	✓	12
Ineva SOFT Child Non Vented **	✓	✓	✓	✓	12
Respireo SOFT Nasal	✓	✓	✓	N.A.	20

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